

Tamper Evident Seals

SCS01B



Introduction:

Case Medical offers a full line of disposables and other ancillary products designed for the SteriTite® and other DIN sized container systems. SteriTite® seals are designed to protect your sealed containers from accidental opening after sterilization and alert you of possible deliberate tampering at any stage.

The SCS01B is used with pre-vacuum, gravity displacement steam and ETO sterilization. The tamper evident seal is manufactured from heat and moisture resistant polypropylene material that is compatible with both steam (high) and low temperature sterilization systems.

The presence of a chemical indicator dot on the tab gives the user an assurance that the set has been processed. Utilize the corresponding load card which includes the same process indicator for monitoring external endpoint response. The indicator on the tab and load card turn from tan to brown when sterilized in an autoclave and from tan to rust orange when exposed to ethylene oxide.

SteriTite tamper evident seals are chemical and temperature resistant for the identified purpose. The pad lock design is not only tamper proof, but safer. All arrow tips when broken upon removal remain encapsulated in the barrel.

Directions for Use:

To Assemble

- Thread the seal arrow through the metal latch of your container
- Push the two sections, arrow and barrel, of the seal to lock

To Remove

- Grasp lower portion of seal, push barrel toward clip
- Twist until the two halves break apart
- Unthread seal from latch and discard
- Do not pull up on the latch to break the seal



SCS01B
1000/pk



© Copyright 2015 Case Medical, Inc.®