

DoseTech™ DMS

Detergent Management System Pre-soak detergent dosing system

The DoseTech DMS is a simple, easy-to-use dosing system that delivers precise amounts of detergent into the sink.

- Precision detergent dosing
- Eliminates variability of bottle mounted pumps
- Out-of-temperature and detergent low level alerts
- Usage reports for quality management and surveyor audits



The DoseTech System features built-in programmable settings, allowing users to set dosing parameters that will meet specific conditions for removing soil at various sink levels for achieving optimum cleaning results.


Bring consistency to your decontamination process with dosing validation, a reliable peristaltic pump, and handy dual soak timer.



How the DoseTech System Works

- A precision flow meter provides optimal dosing accuracy.
- The unit alerts the operator to out-of-chemistry conditions (optional Low Level Alarm).
- A visual confirmation appears on the display of the system so users know that the correct amount of chemistry has been dosed.
- Built-in software logs date, time, chemical usage, temperature, and errors.
- When a system cannot be mounted directly over the cleaning sink, the remote control allows the user to control the unit from a distance of up to 12 feet away.

Get automated detergent dosing with the touch of a button with the DoseTech™ DMS pre-soak detergent dosing system.

Product Specifications	
Dimensions	6" H x 7" W x 6-1/4" D 15.2 cm x 17.8 cm x 15.9 cm
Case Rating	IP-64
Dosing Pump Flow Rate	250 ml/min.
Power Supply/Voltage	Wall Mount Type, IN: 110-240 VAC 1A 50-60 Hz Out: 24 VDC 1.67A
Chemical Compatibility – Dose Pump	Industry standard enzymatic and other detergents.
Unit Weight	4.3 lbs, 1.95 kg
Temperature Sensor	Max temperature = 257° F, 125° C
Approvals - Pending	CE 



Ordering information

Part ID	Description	Per Case
CSDMS01	DoseTech™ DMS with temperature sensor	1
CSDMS02	DoseTech™ DMS with flow meter and temperature sensor	1
CSDMS03	DoseTech™ DMS with USB port, flow meter, temperature sensor	1
	Accessory - Optional sink top mounting kit	1
	Accessory - Optional foot pedal control unit	1
	Accessory - Optional low level alarm	1

Case Medical, Inc.
19 Empire Blvd
South Hackensack, NJ 07606
U.S. and Canada 1-888-227-CASE
Worldwide +1 201-313-1999
www.casemed.com

DoseTech is a trademark of Knight LLC.
Copyright © 2019 by Case Medical, Inc.
All rights reserved. Rev A 5/19



For more information contact
info@casemed.com
Phone: 888-227-CASE (2273)
www.casemed.com