

# Case Solutions®

## Surface Cleaning Instruments Brushes



### Introduction:

Case Solutions® instrument cleaning brushes are ideal for cleaning hard to reach surfaces and crevices of surgical devices. Our surface cleaning brushes have nylon bristles and secure plastic handle grips. Included with the following brush kits and sold in packs of 12.



**BRD07DZ**



**BRS07DZ**

Case Solutions brushes are also available in a variety of sizes and organized in specialty kits as follows:

CSIBK1 GENERAL			CSIBK2 MICRO/EYE		
● Red	BR5F-06DZ	Brush 6" long, 5 french/1.67 mm	None	BR3F-06DZ	Brush 6" long, 3 french/1 mm
○ White	BR6F-08DZ	Brush 8" 6 french/2 mm	● Red	BR5F-06DZ	Brush 6" 5 french/1.67 mm
● Black	BR8F-08DZ	Brush 8" long, 8 french/2.6 mm	○ White	BR6F-08DZ	Brush 8" long, 6 french/2 mm
● Pink	BR10F-08DZ	Brush 8" long 10 french, 3.3 mm	● Black	BR8F-08DZ	Brush 8" long 8 french, 2.6 mm
● Orange	BR12F-08DZ	Brush 8" long 12 french, 4 mm	● Pink	BR10F-08DZ	Brush 8" long 10 french, 3.3 mm
○ White	BR03-12DZ	Twisted Wire 12" long, 3 mm	● Orange	BR12F-08DZ	Brush 8" long 12 french, 4 mm
● Green	BR05-16DZ	Twisted Wire 16" long, 5 mm	● Red	BR02-12DZ	Twisted Wire 12" long, 2 mm
● Yellow	BR07-16DZ	Twisted Wire 16" long, 7 mm	○ White	BR03-12DZ	Twisted Wire 12" long, 3 mm
● Pink	BR10-24DZ	Twisted Wire 24" long, 10 mm	None	BRD07DZ	Plastic Handle 7" Double Ended Brush
None	BRD07DZ	Plastic Handle 7" Double Ended Brush			
None	BRS07DZ	Plastic Handle 7" Single Ended Brush			
CSIBK3 ENDOSCOPY					
● Red	BR02-12DZ	Twisted Wire 12" long, 2 mm	● Orange	BR2.5-24DZ	Twisted Wire 24" long, 2.5 mm
○ White	BR03-12DZ	Twisted Wire 12" long, 3 mm	● Green	BR05-24DZ	Twisted Wire 24" long, 5 mm
● Green	BR05-12DZ	Twisted Wire 12" long, 5 mm	● Pink	BR10-24DZ	Twisted Wire 24" long, 10 mm
● Green	BR05-16DZ	Twisted Wire 16" long, 5 mm	● Green	BR05-16DZ	Twisted Wire 16" long, 5 mm
● Pink	BR10-16DZ	Twisted Wire 16" long, 10 mm	● Pink	BR10-16DZ	Twisted Wire 16" long, 10 mm
None	BR2.5-39DZ	Twisted Wire with nylon sleeve 39" long, 6 french/2.5mm			

Note: The color of the brush identification bead is related to the diameter of the brush. The last two digits of the part number provide the length.

### Directions for Use:

- Case Solutions brushes are designed to be reused or disposed of after single use based on health care policies and procedures as well as manufacturer's IFU
- Follow your device manufacturers' IFU and hospital policies for reprocessing reusable devices
- Choose cleaning media of choice and follow instructions for correct dilution and rinsing
- Reusable brushes should be cleaned between each use and disinfected daily
- Inspect brushes prior to use, if any exhibit visible soil, wear, or fraying, discard and replace
- If debris is observed, submerge the end of the brush in the cleaning solution and physically remove the debris
- Repeat until no debris is observed on the brush
- Brushes may be cleaned in automated washers secured in a basket or perforated tray utilizing pH neutral detergent or enzymatic detergent
- Residual detergent should be thoroughly rinsed from the brush
- Inspect prior to next use

Note: Reusable brushes can create a risk for cross contamination.

© Copyright 2017 Case Medical, Inc.® Revision A

# Case Solutions®


## Endoscopy Cleaning Brushes



### Directions for Endoscopy Use:

- The use of cleaning brushes should be in compliance with the device manufacturer's IFU
- Case Medical recommends that endoscope cleaning brushes are for single-use only. They should not be reused or reprocessed
- Endoscope brushes are used in association with water and detergent to remove soil and bioburden from the channels and valves of flexible endoscopes
- The use of cleaning brushes should always be in compliance with the device manufacturer's IFU
- Select the proper diameter and length of brush for the internal surface/channel to be cleaned.
- Always brush the channel while the endoscope is submerged in the cleaning solution
- Do not force a brush into a channel to avoid possible damage to the endoscope
- Carefully advance the brush completely through the channel, using short strokes, until it emerges from the distal end of the scope
- Dispose of the brush after use, according to your facility biohazard disposal policy

<b>BR2.5-39DZ</b>	Brush nylon wire with sleeve 39", 6 french/2.5mm	12 per pack
-------------------	--	-------------

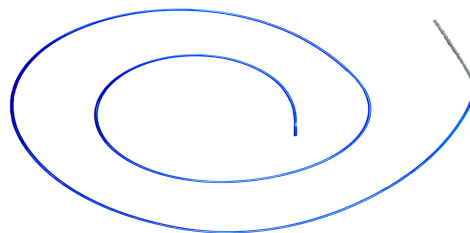

**BR02-39DZ**

**For more information please contact Case Medical at 888-227-CASE or 855-360-8787**

© Copyright 2017 Case Medical, Inc.® Revision A