


# SteriTite®

## Single Use Consumables Kits

For use with SteriTite® containers or other DIN-sized container systems, the SteriTite Single Use Consumables Kits allow you to simplify ordering by combining these three consumable items in just one SKU. Available in various configurations, you can choose kits for Steam and EtO or kits for low temperature H<sub>2</sub>O<sub>2</sub> sterilization.

### Kits for Steam and EtO Sterilization

- **Part ID SCKIT1BP** for standard containers includes:
  - Standard round paper filters (SCF01) 3 pks (500 ea/pk)
  - Blue tamper evident seals (SCS01B) 1 pk (500 ea/pk)
  - Load cards (SCL01) 1 pk (500 ea/pk)

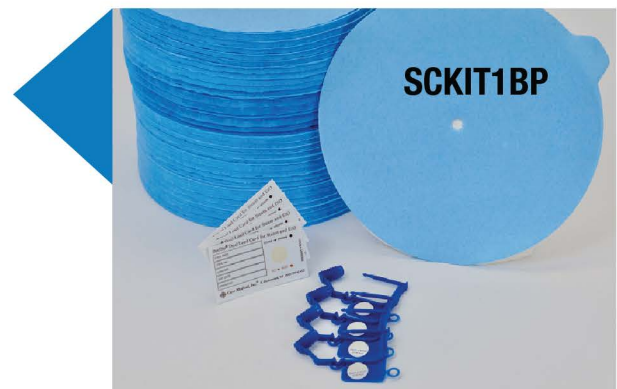

- **Part ID SCKIT2BP** for mini and narrow containers includes:
  - Rectangular paper filters (SCFM01) 1 pk (500 ea/pk)
  - Blue tamper evident seals (SCS01B) 1 pk (500 ea/pk)
  - Load cards (SCL02) 1 pk (500 ea/pk)

### Kits for H<sub>2</sub>O<sub>2</sub> Sterilization

- **Part ID SCKIT1WN** for standard containers includes:
  - Standard round polypro filters (SCF02) 3 pks (500 ea/pk)
  - White tamper evident seals (SCS01W) 1 pk (500 ea/pk)
  - Load cards (SCLH2023) 1 pk (500 ea/pk)



- **Part ID SCKIT2WN** for mini and narrow containers includes:
  - Rectangular polypro filters (SCFM02) 1 pk (500 ea/pk)
  - White tamper evident seals (SCS01W) 1 pk (500 ea/pk)
  - Load cards (SCLH2024) 1 pk (500 ea/pk)

**Note:** SteriTite containers are cleared for use in all sterilization modalities. Polypropylene filters are required for H<sub>2</sub>O<sub>2</sub> sterilization cycles and may be used in Steam and EtO cycles. Paper filters can be used in Steam and EtO cycles only. To achieve 5 to 8 minute (minimum) dry time in pre-vac steam cycles, use paper filters.

## Filters

The SteriTite® universal sealed container system uses filters to establish and maintain a bacterial barrier, protecting the processed instrument sets from contamination from the environment.

### Directions for Use:

- Assemble for use. Place a filter over each vented area.
- Secure with the associated filter retention plate. Lock in place.
- Filters are supplied non-sterile and for single use only.
- Discard after each use.

## Tamper Evident Seals

Tamper evident seals provide visual assurance that the set has not been opened. Use with SteriTite containers or other DIN-sized containers.

### Directions for Use:

- Thread the seal through the metal loop on your container's latch.
- Push the two sections of the seal together, locking the seal.
- To remove: Grasp lower position of seal, push barrel toward clip and twist until the seal breaks apart.
- Unthread from latch and discard.
- Do not pull up on the latch to break the seal.

## Load Cards

Load cards provide space to record load information and include an external color-change process indicator to show that the set was processed.

### Directions for Use:

- Record load information in space provided and slide into dedicated slot on container.
- Color change indicates the set that has been exposed to the specific sterilization process.



With the convenience of a kit, order just one SKU and you will have all the components on hand to ensure that instrument sets have been properly processed and are ready for use for patient procedures.

SCKIT1BP	SteriTite® Consumables Kit, Steam and EtO, standard containers
SCKIT2BP	SteriTite® Consumables Kit, Steam and EtO, mini and narrow containers
SCKIT1WN	SteriTite® Consumables Kit, H <sub>2</sub> O <sub>2</sub> , standard containers
SCKIT2WN	SteriTite® Consumables Kit, H <sub>2</sub> O <sub>2</sub> , mini and narrow containers

Case Medical Inc.  
50 West Street  
Bloomfield, NJ 07003  
U.S. and Canada 1-888-227- CASE  
Worldwide +1 201-333-1999  
www.casemed.com

SteriTite is a registered trademark of Case Medical.  
Copyright © 2023 Case Medical Inc.  
All rights reserved. Rev C 11/23